



# **Lab** help

#### Cosmetic & Full Mouth Checklist for Success

If you have any questions or need assistance you can contact us at 800-336-3053 or visit our website and click on the icon as shown to the right, which you can find along the bottom right hand corner of our website at burbankdental.com. Just request Lab Help and we will be glad to assist you.

Also, check out our Resources section when visiting our website.



# 1 First Appointment

- 1 Pre-Op Models Photos
- 1 Full face smile (forehead to chin), Lip at rest close-up, Smile close-up
- 2 Incisal Shade Picture (Shade tabs(s) incisal edge to incisal edge on same vertical plane.)
- 3 Over All shade Picture (Shade tab next to target shade tooth, oriented body-body, incisal to incisal)









#### 2 Proposed Plan

- 1 Steps
- 1 Quantity, tooth #'s, and pontics
- 2 Tissue or other augmentations planned
- 3 Staging plan
- 4 VDO change (quantify) or remain as current
- (5) Type of restorative material(s)
- 6 Patient concerns and goals for case
- 7 Any other goals or concerns
- 2 Second Appointment
  - Lab Delivers: Diagnostic
    Wax-Up, Plan Verification
    Report, Putty Stent of Wax-Up,
    Prep Guide Model & Prep-Stent



- Second Appointment: Send to Lab
- Rx for Dura-Temps & VDO Bite

### Third Appointment



Lab Delivers: DuraTemps


3 "Sections" staging of preparation & bite records



- 1 Prep single quadrant take hard bite Luxabite®
- 2 Prep second quadrant take hard bite Luxabite®
- 3 3rd and final quadrant place previous hard bites and take 3rd hard bite





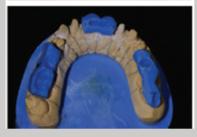









# **Lab** help

Cosmetic & Full Mouth Checklist for Success

Page 2



# Third Appointment Cont.

- 2b If VDO change
- 1 Prep single section, try-in pre-made lab provisionals., reline Check VDO Relationship.
- 2 Repeat for 2nd and 3rd section.
- 3 Take 3 bites each with 2 provisional sections in place. (eg. Anterior bite with both posterior section provisionals in place.)
- 2c If both arches are being restored
- 1 Prep both maxillary & mandibular anterior sections.
  Try-in & reline DuraTemps, verify VDO and occlusal relationship.
- 2 Prep one posterior section, both maxillary and mandibular.
  With anterior provisionals in place.
  Try-in & reline DuraTemps, verify VDO and occlusal relationship.

Repeat for contra-lateral section.

- 3 Take 3 bites for EACH section. (With provisionals seated on other 2 sections.)
  - a. Prep to prep.
  - b. (2) Prep to opposing temps.
- 4 Check All 3 Prep to Prep bites. All 3 should fit easily and passively. If not, find the 2 that do and retake 3rd.

# 4 Fabrication of Final Restorations

- Send Rx, Impressions & Bites to Lab
- 1 Send to lab:
  - Full arch prep impressions
  - Impressions of Provisionals
  - · All bites in labeled baggies
- Provisional Approval & Records
- 1 Photos & models of approved provisionals
- 2 Send Rx, Provisional Models and Photos to Lab



- 3 Lab Delivers: Bisque bake for try-in.
- Fifth Appointment:
  Try-in and adjust bisque bake.



Rx for final finish of restorations

- 5 Sixth Appointment Delivery of finished restorations
- Lab Delivers:
  Final restorations
- 2 Delivery of restorations





